# REDULAT

*Libera*

## MIKROKLISTIERE

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## MICRO ENEMAS

**Micro enemas containing glycerol, chamomile and mallow 9g**

**Indication:**  
Short-term treatment of occasional constipation.

**Mechanism of action:**  
Mobilises and lubricates the faecal mass, facilitating evacuation with emollient effect.

**Composition:**  
1 micro enema contains 6,75 g glycerol; excipients: pregelatinised potato starch, chamomile fluid extract, mallow fluid extract, purified water.

**Contraindications:**  
Micro enemas must not be used in the event of acute abdominal pain or that of an unknown origin.

**Side effects:**  
In particularly sensitive patients or those with a previously inflamed mucous membrane, irritation of the rectal mucosa may occur.

**Warning:**  
No data are available on the use of the medical device during pregnancy and breastfeeding. Ask your doctor before taking this medical device during pregnancy and breastfeeding.  
Do not use in case of incompatibility with any component of the preparation.  
Do not use the product if the packaging is damaged or after the best before date indicated on the pack. Expiration date indicated refers to the product in intact packaging, correctly stored.

**Dose:**  
Adults: one single-dose micro-enema as needed, for a maximum of 1 or 2 administrations a day.

**Instruction for use:**  
**RECTAL USE.**  
Each single-dose micro enema of GLYCEROL CHAMOMILE AND MALLOW is individually sealed until ready to use; this ensures the hygiene and ease of use. Remove the cap on the single-dose container. It may be useful to lubricate the applicator with a drop of the solution itself, before introducing it into the rectum and squeezing the container. Remove the applicator holding the container squeezed. Each container must be used for one administration only. All product residues must be eliminated.

**Storage:**  
Micro enemas must be stored in a dry place away from heat sources..

KEEP THE PRODUCT OUT OF THE REACH OF CHILDREN.

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